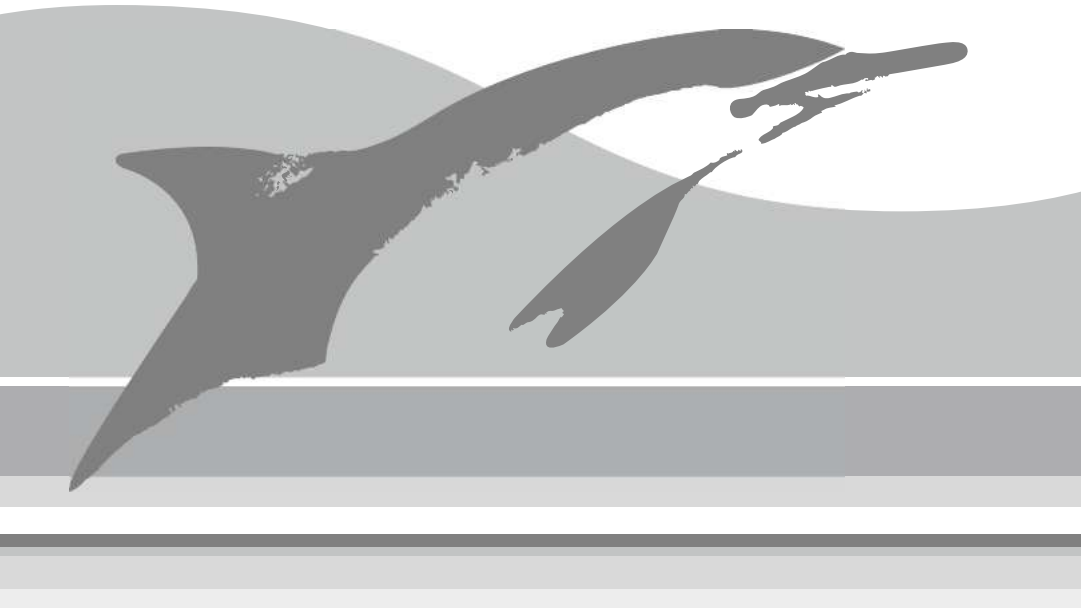
Heraeus

PALAMIX®

Vacuum mixing system with collection under vacuum

PALAMIX®

EN	Instructions for use (Revision status: 2020-11)	4
EN (US)	Note: Instructions for use for sale within the US. Instructions for use (Revision status: 2020-11)	9
BG	Инструкции за употреба (Дата на актуализиране на текста: 2020-11)	14
CS	Návod k použití (Poslední revise: 2020-11)	19
DA	Brugsanvisning (Aktualiseret: 2020-11)	24
DE	Gebrauchsanweisung (Revisionsstand: 2020-11)	29
EL	Οδηγίες χρήσης (Ημερομηνία αναθεώρησης: 2020-11)	34
ES	Instrucciones de uso (Última revisión: 2020-11)	39
ET	Kasutusjuhend (Viimati muudetud: 2020-11)	44
FI	Käyttöohje (Revisiostatus: 2020-11)	49
FR	Notice d'utilisation (Date de révision: 2020-11)	54
HR	Upute za uporabu (Stanje revizije: 2020-11)	59
HU	Alkalmazási utasítás (Aktualizálás időpontja: 2020-11)	64
IT	Istruzioni per l'uso (Ultima revisione: 2020-11)	69
KK	Пайдалану нұсқаулары (Редакция күйі: 2020-11)	74
KO	사용 설명서 (2020년 11월 수정)	79
LT	Naudojimo instrukcija (Tikrinta: 2020-11)	84
LV	Lietošanas instrukcija (Pārbaudes statuss: 2020-11)	89

NL	Gebruiksaanwijzing (Revisiestatus: 2020-11)	94
NO	Bruksinformasjon (Revisjonsstatus: 2020-11)	99
PL	Instrukcja użytkowania (Ostatnia aktualizacja: 2020-11)	104
PT	Instruções de utilização (Data da última actualização: 2020-11)	109
RO	Instrucțiuni de utilizare (Stadiu revizie: 2020-11)	114
RU	Инструкция по применению (Ревизионный статус: 2020-11)	119
SK	Návod na použitie (Posledná revízia textu: 2020-11)	124
SL	Navodilo za uporabo (Datum revizije: 2020-11)	129
SV	Bruksanvisning (Revision status: 2020-11)	134
TH	คำแนะนำวิธีใช้ (แก้ไขปรับปรุงครั้งสุดท้ายเมื่อ: 2020-11)	139
TR	Kullanma Talimatı (Bilgilerin son güncellenme tarihi: 2020-11)	144
ZH	使用说明书 (修订状态: 2020 年 11 月)	第149页

General information

PALAMIX® is an orthopaedic vacuum mixing and application system for bone cements.

PALAMIX® is available in two sizes: **PALAMIX® uno** and **PALAMIX® duo** and they are suitable for automatic collection of bone cement within the mixing cartridge under vacuum.

Components

PALAMIX® uno (details for **PALAMIX® duo** in brackets) contains the following sterile components:

- 1 (2) mixing cartridge, large enough to hold up to two (four) pouches of Heraeus Medical GmbH bone cement (uno: 2x40, duo: 4x40)
- 1 (2) mixing rod with integrated mixing paddle and nozzle
- 1 (1) funnel
- 1 (1) vacuum hose
- 1 (1) femoral pressuriser
- 1 (1) knee pressuriser
- 1 (2) short nozzle
- 1 (2) spatula clip
- 1 (1) work tray

All components are not made with natural rubber latex and all components that come into contact with bone cement are not made with PVC.

Intended purpose

PALAMIX® is intended for in mixing bone cement with different viscosities under vacuum.

Indications

PALAMIX® is indicated for use in mixing bone cement with different viscosities under vacuum.

Contraindications

No contraindications for the use of **PALAMIX® uno** and **PALAMIX® duo** are known.

Adverse events

No adverse events for the use of **PALAMIX® uno** and **PALAMIX® duo** are known.

Intended Patient Population

Patients requiring bone cement application during surgery.

Intended User

Healthcare professionals in a clinical context and experienced in the handling of the product.

Warnings

PALAMIX® uno and **PALAMIX® duo** are single-use devices and must never be re-used! Re-use may result in diminished safety, performance, and compliance with relevant specifications.

Precautions**Bone cement**

The sequence described in these instructions is optimized to suit Heraeus bone cements. If you are using bone cements from other manufacturers, please observe their specifications, the sequence for filling the vacuum mixing systems and their instructions for use.

Regarding intended users

Read these instructions for use before using for the first time and familiarize yourself with operation of the system. Test the bone cement properties and handling of **PALAMIX®** beforehand to account for your specific conditions.

The bone cements should be extruded as soon as the surface of the bone cement is no longer sticky.

The "finger test" should be carried out to check this. However, the bone cement must be extruded at the latest by the end of the temperature-dependent application time specified by the manufacturer (see corresponding instructions for use for the particular bone cement). During use, ensure that a suitable distance is left between non-sterile products and the sterile products **PALAMIX® uno** and **PALAMIX® duo**.

The amount of bone cement dough required depends on the specific surgical intervention and on the technique being used. At least one additional set of **PALAMIX® uno** or **PALAMIX® duo** should be available before commencing the operation.

Supporting equipment

Supporting equipment is required for the use of PALAMIX®:

Product	Description	Quantity per sales unit	Article number
PALAMIX® cement gun	Reusable application gun	1	66036163
PALAMIX® vacuum pump	Vacuum pump with non-return valve	1	66036748
Optional			
PALAMIX® slim nozzle	For use with low-viscosity bone cements; \varnothing 7 mm	10	66036747
PALAMIX® medium nozzle	Flexible, conical nozzle; \varnothing 8.7 – 12.6 mm	10	66043960

The instructions for use of the supporting equipment must be followed.

Shelf life and sterilization

Shelf life

The shelf life of **PALAMIX® uno** or **PALAMIX® duo** is printed on the package. Do not use **PALAMIX® uno** or **PALAMIX® duo** if the date indicated has expired.

Sterilization

PALAMIX® uno and **PALAMIX® duo** have been sterilized using ethylene oxide. The product must not be resterilized.

Storage

PALAMIX® uno and **PALAMIX® duo** must be stored under dry and clean conditions.

Care shall be exercised during transport and handling of **PALAMIX® uno** and **PALAMIX® duo** to avoid any damage or alteration to the performance characteristics and its packaging as received.

Do not remove **PALAMIX® uno** or **PALAMIX® duo** from the sterile packaging until immediately before use. Do not use if packaging is damaged.

Should you require further information, your Heraeus Medical GmbH partners would be pleased to assist you.


Disposal

Single components of **PALAMIX® uno** and **PALAMIX® duo** as well as packaging material must be disposed of as clinical waste in compliance with local regulations.


Product overview

Product	Description	Quantity per sales unit	Article number
PALAMIX® uno	Vacuum mixing system with collection under vacuum	10	66057893
PALAMIX® duo	Vacuum mixing system with collection under vacuum, with two cartridges	10	66057897

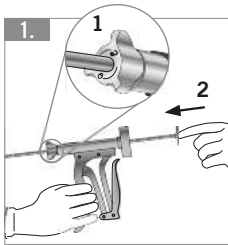
Operating instructions

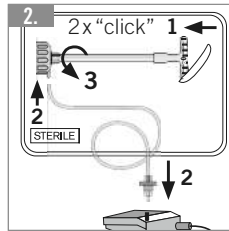1. Release the locking mechanism (↺).
2. Push the feed rod back to the stop.

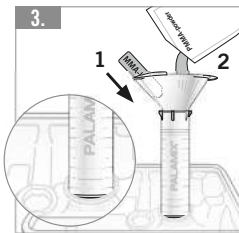
1. Close the handle on the mixing rod (2x "click"!).
2. Connect the vacuum hose to the cartridge and **PALAMIX® vacuum pump**.
3. **PALAMIX®** is supplied with the locking screw opened. This setting must not be changed for the mixing process.


For filling, the cartridge must be positioned in tray station "1". When using Heraeus Medical bone cements, the monomer liquid must be added first, followed by the bone cement powder.

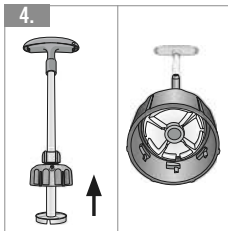
DUO: Set the funnel aside in the sterile area.

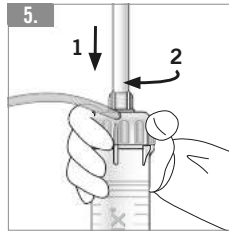
Retract the mixing rod as far as the stop before locking the cap with the cartridge.

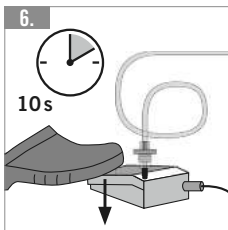
Place the cap on the cartridge and screw on tightly until it stops, so that the ribs on the cap and the bars on the cartridge are aligned one above the other.

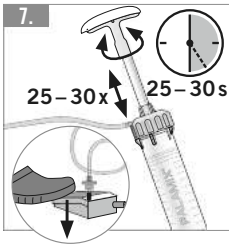
After closing the cartridge and before mixing, activate the **PALAMIX® vacuum pump** with your foot for 10 seconds. The vacuum generated can be seen in the cartridge.

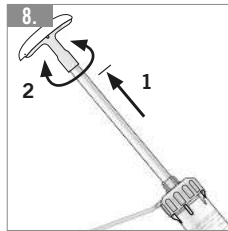
Mix the bone cement by steadily moving the mixing rod up and down the entire length of the cartridge for at least 25–30 strokes (1 stroke per second).^{*} During the entire mixing time activate the **PALAMIX® vacuum pump**.

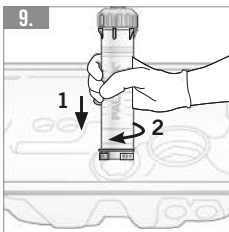
1. Retract the mixing rod as far as the stop and
2. rotate briefly to the right/left.

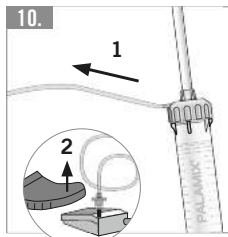
1. Hold the cartridge on the stem and insert it into tray station "2" without applying force.
2. Rotate the cartridge. When rotating, hold the stem of the cartridge, not the cap. The cartridge base is released and moves rapidly upward under vacuum. The bone cement is collected in the cartridge.

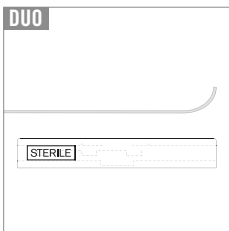1. Remove the vacuum hose.
2. Release the **PALAMIX® vacuum pump** footswitch.

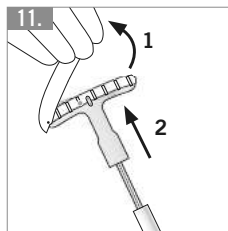
DUO: Set the vacuum hose aside in the sterile area.


1. Release the handle,
2. retract the mixing rod and set it aside in the sterile area.

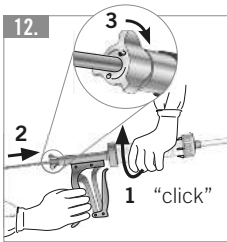
^{*} For exact mixing times, refer to the instructions for use of the bone cement manufacturer.

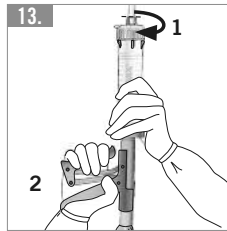
1. Screw the cartridge onto the **PALAMIX® cement gun** with a quick rotational movement ("click"!).
2. Push the feed rod into the cartridge as far as it will go.
3. Secure the locking mechanism (🔒).

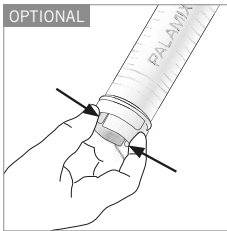
1. Tighten the locking screw on the nozzle beneath the predetermined breaking point.
2. Advance the bone cement up to the nozzle tip. If there is no adhesion, apply.

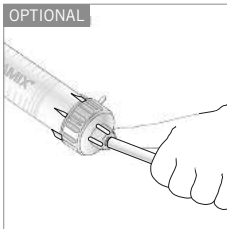
If activation under vacuum on the tray station is not possible, it can be generated manually. Two small tabs on the cartridge base must be pressed by hand. The cartridge base moves rapidly upward under vacuum. The bone cement is collected in the cartridge.

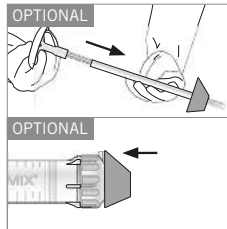
Break off the nozzle at the predetermined breaking point and remove the locking screw.

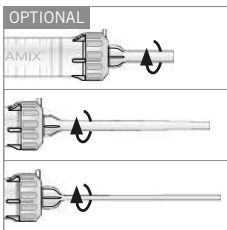Expel the bone cement remaining in the nozzle with the aid of the mixing rod. Push on the femoral pressuriser, as necessary.

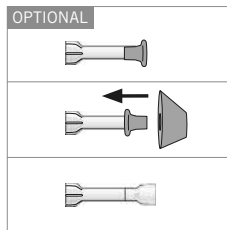Screw on the short nozzle (e.g. for application in the knee), **PALAMIX® medium nozzle** (e.g. for minimal-invasive procedures) or **PALAMIX® slim nozzle** (for use with low-viscosity bone cements).

Short nozzle with knee pressuriser.

Short nozzle with knee pressuriser and femoral pressuriser.

Short nozzle with spatula clip.

General information

PALAMIX® is an orthopaedic vacuum mixing and application system for bone cements.

PALAMIX® is available in two sizes: **PALAMIX® uno** and **PALAMIX® duo** and they are suitable for automatic collection of bone cement within the mixing cartridge under vacuum.

Components

PALAMIX® uno (details for **PALAMIX® duo** in brackets) contains the following sterile components:

- 1 (2) mixing cartridge, large enough to hold up to two (four) pouches of Heraeus Medical GmbH bone cement (uno: 2 x 40, duo: 4 x 40)
- 1 (2) mixing rod with integrated mixing paddle and nozzle
- 1 (1) funnel
- 1 (1) vacuum hose
- 1 (1) femoral pressuriser
- 1 (1) knee pressuriser
- 1 (2) short nozzle
- 1 (2) spatula clip
- 1 (1) work tray

All components are not made with natural rubber latex and all components that come into contact with bone cement are not made with PVC.

Intended purpose

PALAMIX® is intended for in mixing bone cement with different viscosities under vacuum.

Indications

PALAMIX® is indicated for use in mixing bone cement with different viscosities under vacuum.

Contraindications

No contraindications for the use of **PALAMIX® uno** and **PALAMIX® duo** are known.

Adverse events

No adverse events for the use of **PALAMIX® uno** and **PALAMIX® duo** are known.

Intended Patient Population

Patients requiring bone cement application during surgery.

Intended User

Healthcare professionals in a clinical context and experienced in the handling of the product.

Warnings

PALAMIX® uno and **PALAMIX® duo** are single-use devices and must never be re-used! Re-use may result in diminished safety, performance, and compliance with relevant specifications.

Precautions**Bone cement**

The sequence described in these instructions is optimized to suit Heraeus bone cements. If you are using bone cements from other manufacturers, please observe their specifications, the sequence for filling the vacuum mixing systems and their instructions for use.

Regarding intended users

Read these instructions for use before using for the first time and familiarize yourself with operation of the system. Test the bone cement properties and handling of **PALAMIX®** beforehand to account for your specific conditions.

The bone cements should be extruded as soon as the surface of the bone cement is no longer sticky.

The "finger test" should be carried out to check this. However, the bone cement must be extruded at the latest by the end of the temperature-dependent application time specified by the manufacturer (see corresponding instructions for use for the particular bone cement). During use, ensure that a suitable distance is left between non-sterile products and the sterile products **PALAMIX® uno** and **PALAMIX® duo**.

The amount of bone cement dough required depends on the specific surgical intervention and on the technique being used. At least one additional set of **PALAMIX® uno** or **PALAMIX® duo** should be available before commencing the operation.

Supporting equipment

Supporting equipment is required for the use of PALAMIX®:

Product	Description	Quantity per sales unit	Article number
PALAMIX® cement gun	Reusable application gun	1	66036163
PALAMIX® vacuum pump	Vacuum pump with non-return valve	1	66036748
Optional			
PALAMIX® slim nozzle	For use with low-viscosity bone cements; Ø 7 mm	10	66036747
PALAMIX® medium nozzle	Flexible, conical nozzle; Ø 8.7–12.6 mm	10	66043960

The instructions for use of the supporting equipment must be followed.

Shelf life and sterilization

Shelf life

The shelf life of **PALAMIX® uno** or **PALAMIX® duo** is printed on the package. Do not use **PALAMIX® uno** or **PALAMIX® duo** if the date indicated has expired.

Sterilization

PALAMIX® uno and **PALAMIX® duo** have been sterilized using ethylene oxide. The product must not be resterilized.

Storage

PALAMIX® uno and **PALAMIX® duo** must be stored under dry and clean conditions.

Care shall be exercised during transport and handling of **PALAMIX® uno** and **PALAMIX® duo** to avoid any damage or alteration to the performance characteristics and its packaging as received.

Do not remove **PALAMIX® uno** or **PALAMIX® duo** from the sterile packaging until immediately before use. Do not use if packaging is damaged.

Should you require further information, your Heraeus Medical GmbH partners would be pleased to assist you.


Disposal

Single components of **PALAMIX® uno** and **PALAMIX® duo** as well as packaging material must be disposed of as clinical waste in compliance with local regulations.


Product overview

Product	Description	Quantity per sales unit	Article number
PALAMIX® uno	Vacuum mixing system with collection under vacuum	10	66057893
PALAMIX® duo	Vacuum mixing system with collection under vacuum, with two cartridges	10	66057897

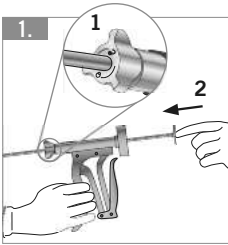
Operating instructions

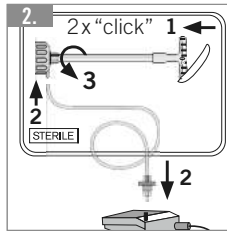1. Release the locking mechanism (↺).
2. Push the feed rod back to the stop.

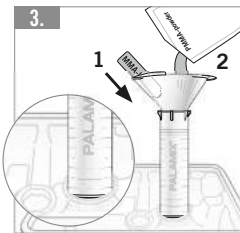
1. Close the handle on the mixing rod (2x "click"!).
2. Connect the vacuum hose to the cartridge and **PALAMIX® vacuum pump**.
3. **PALAMIX®** is supplied with the locking screw opened. This setting must not be changed for the mixing process.


For filling, the cartridge must be positioned in tray station "1". When using Heraeus Medical bone cements, the monomer liquid must be added first, followed by the bone cement powder.

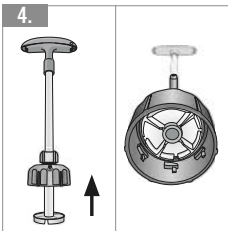
DUO: Set the funnel aside in the sterile area.

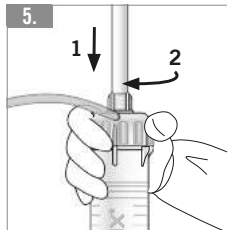
Retract the mixing rod as far as the stop before locking the cap with the cartridge.

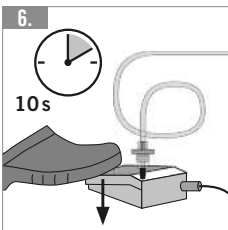
Place the cap on the cartridge and screw on tightly until it stops, so that the ribs on the cap and the bars on the cartridge are aligned one above the other.

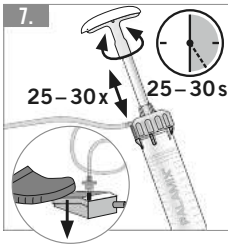
After closing the cartridge and before mixing, activate the **PALAMIX® vacuum pump** with your foot for 10 seconds. The vacuum generated can be seen in the cartridge.

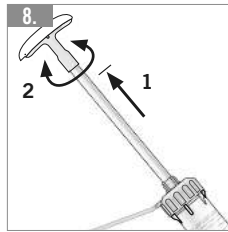
Mix the bone cement by steadily moving the mixing rod up and down the entire length of the cartridge for at least 25–30 strokes (1 stroke per second).^{*} During the entire mixing time activate the **PALAMIX® vacuum pump**.

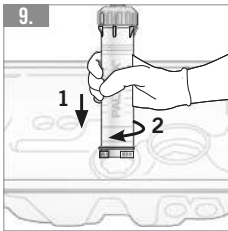
1. Retract the mixing rod as far as the stop and
2. rotate briefly to the right/left.

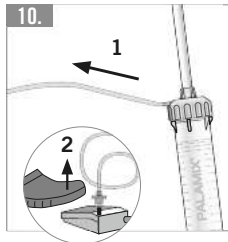
1. Hold the cartridge on the stem and insert it into tray station "2" without applying force.
2. Rotate the cartridge. When rotating, hold the stem of the cartridge, not the cap. The cartridge base is released and moves rapidly upward under vacuum. The bone cement is collected in the cartridge.

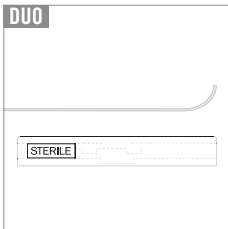1. Remove the vacuum hose.
2. Release the **PALAMIX® vacuum pump** footswitch.

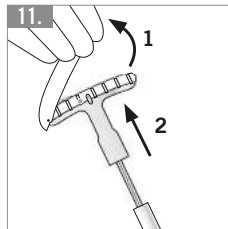
DUO: Set the vacuum hose aside in the sterile area.

1. Release the handle,
2. retract the mixing rod and set it aside in the sterile area.

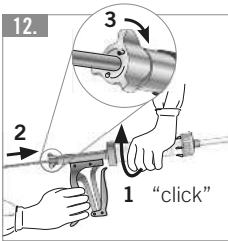
^{*} For exact mixing times, refer to the instructions for use of the bone cement manufacturer.

1. Screw the cartridge onto the **PALAMIX® cement gun** with a quick rotational movement ("click"!).


2. Push the feed rod into the cartridge as far as it will go.

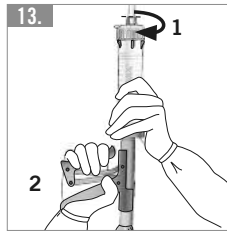3. Secure the locking mechanism (🔒).


1. Tighten the locking screw on the nozzle beneath the predetermined breaking point.

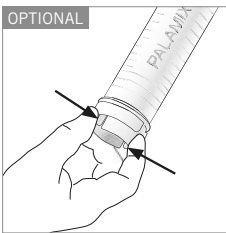
2. Advance the bone cement up to the nozzle tip. If there is no adhesion, apply.

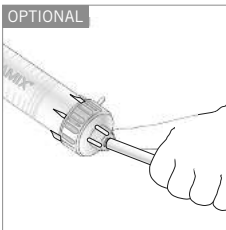
If activation under vacuum on the tray station is not possible, it can be generated manually. Two small tabs on the cartridge base must be pressed by hand. The cartridge base moves rapidly upward under vacuum. The bone cement is collected in the cartridge.

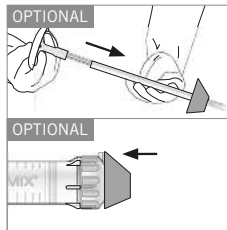
Break off the nozzle at the predetermined breaking point and remove the locking screw.

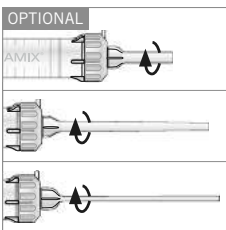Expel the bone cement remaining in the nozzle with the aid of the mixing rod. Push on the femoral pressuriser, as necessary.

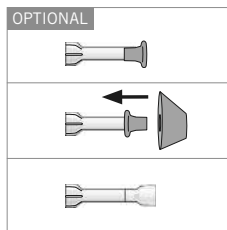Screw on the short nozzle (e.g. for application in the knee), **PALAMIX® medium nozzle** (e.g. for minimal-invasive procedures) or **PALAMIX® slim nozzle** (for use with low-viscosity bone cements).

Short nozzle with knee pressuriser.

Short nozzle with knee pressuriser and femoral pressuriser.

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PALAMIX® is a trademark of Heraeus Medical GmbH. Made in Germany.